

glue-u
adhesives



APPLICATION
instructions

*PRE FIX,
STANDARD
WITH INSERTS*



PREPARATION

GLUSHU

PREPARATION

GLUSHU

You can reshape the shoe as you would do with a normal aluminium horseshoe.

Good hoof prep is critical to the shoe remaining secure for the complete trim cycle 6-8 weeks.




1x Glushu



1x Shucleaner

1

25-30C
75 -85F

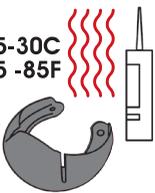

Trim the horse's hoof as normal. Make sure the glue and shoes are warm. 25 - 30 C (75 - 85F) In cold climates a hot box can be used.

2




Wire-brush the hoof, sole and frog to remove all dirt and debris. This will enable a better glue bond.

3



With the edge of a rasp, using downward strokes, rough the surface of the hoof giving special attention to the rear part of the hoof as this has the least gluing surface.

4

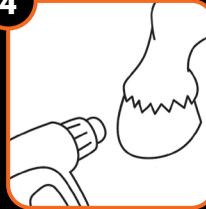

If wet or damp, dry the hoof using a hot air gun or torch. Most glue failure is due to moisture in the hoof.

5



Wire-brush the hoof again to remove any dust. Eventually use the SHU CLEANER to make sure that the hoof will be clean. Do not touch the hoof with your bare hands.

6



Check the fit including the heel for overreach and adjust where necessary. It should be a good fit incl 3mm glue thickness, not loose. You can reshape the shoe as you would do with a normal aluminium horseshoe.

STANDARD APPLICATION


STANDARD APPLICATION

STANDARD APPLICATION METHOD

Standard application for horses.



1x Shubond 150 cc

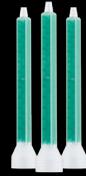

1x Mixingtips



1x Roll hoofwrap



1x Dispenser

1



Express the glue until it is black before you apply. It will only work properly when it extrudes black.

2



Put a medium bead of glue around the inside base of the shoe. Also, put generous beads of glue on the inside surface of the cuffs. Medium bead = approximately 1 cm bead.

3



Wait until glue is activated before applying Glushu to the hoof. Glue becomes warm or colors mat when activated. Holding back the side cuffs, slide the shoe gently onto the hoof, aligning it with the frog and adjust to the correct position. Don't worry.

4




Use wrapping foil to keep the Glushu in position. This application reduces the time to hold up the hoof while the glue is setting. Gently put hoof on the ground. Once glue is set you can remove the wrapping foil and finish the application.

5



Hold the hoof until the glue has completely set. Approx 2-4 minutes. The glue will be quicker in warmer conditions and slower in cooler conditions. It is very important the glue bond is not broken whilst curing.

6

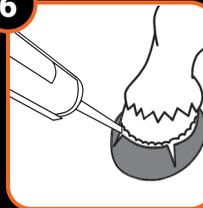

When removing the wrapping foil fill up areas not glued and put a small layer of glue on the edge of the cuff. This to prevent stepping off. Wrap the hoof again until glue is set. Once glue is set and wrapping foil removed finish application using smooth side of hoof rasp.

PRE FIX APPLICATION


PRE FIX APPLICATION

PRE FIXATION METHOD

For horses who cannot hold their feet off the ground due to soreness. For glueing in severe cold circumstances.



1x Shubond 150 cc

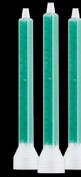

1x Mixingtips



1 x Pre Fix Activator and
1x Pre Fix CA Glue



1x Roll hoofwrap




1x Dispenser

1



Put a medium bead of Pre Fix CA Glue around the base of the hoof.

2

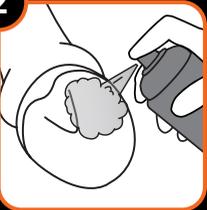

Spray the inside bottom of the shoe with the Pre Fix Activator.

3



Holding back the side cuffs, aligning it with the frog and adjust to the correct position.

Hold for 5 seconds. Once the Pre Fix CA Glue has set, place the hoof on the ground.

4



With the hoof on the ground, pull back the cuffs using a hoof pick and backfill inside the cuff with Shubond.

5



Use hoofwrap to keep the shoe in position while the glue is setting.

6



Place the hoof on the ground and leave for a minimum of 10 minutes before removing the hoof wrap.

REMOVAL

DRAWKNIFE

INSERTS


APPLICATION



1x Draw knife



1x Shubond 150 cc

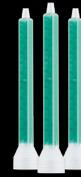

1x Mixingtips



1x Shufix



1x Roll hoofwrap



1x Dispenser

1



Using a draw knife, cut the cuff from the hoof using downward strokes along the glue line.

2



Using nippers, cut the metal shoe from the hoof.

3



Remove any remaining glue or cuff with a rasp or buffer.

1



Apply the insert to the Glushu from above with the glue of your choice.

Glue-U CA glue: 10 seconds
Glue-U Shubond: 5 minutes

TIP! Fill with Glue-U pad material for the best support.

GLUSHU

GLUE-ON HORSEHOE

A FLEXIBLE CUFF BONDED TO A SPECIALLY DESIGNED ALUMINIUM HORSESHOE.

This gives the horse a light, hoof supporting, cushioned shoe which eliminates any stress or damage to the horse's hoof.

CONTACT

GLUE-U[®]ADHESIVES

Glue-U Adhesives BV
Droogdokkeneiland 8
5026SR Tilburg
The Netherlands

+31 (0)13 545 31 18
info@glue-u.com

Glue-U USA Inc
215 Church Street
Georgetown
SC 29440 USA

+1 (313) 509 - 7766
www.glue-u.com

**PRE FIX
APPLICATION
VIDEO**



**STANDARD
APPLICATION
VIDEO**



**MODIFICATION
VIDEO**

